

## EasyWare Version History

This document lists all releases of the EasyOne PC software called EasyWare. Some versions haven't been distributed or only for selected customers. The changes though are cumulative, that means that changes introduced in a version are also included in later versions.

Version	Description
<b>EasyWare</b> <b>EasyPrint</b> <b>V2.25.0.0</b> <b>Rev. B</b>  14.06.2013  <b>Available on</b> <a href="http://www.ndd.ch">www.ndd.ch</a> + CD	<ul style="list-style-type: none"> <li>• Same EasyWare as Rev. A</li> <li>• With full driver support also for Windows 8</li> </ul>
<b>V2.25.0.0</b> 31.01.2013  <b>EasyWare 2013</b> <b>EasyPrint 2013</b> <b>Rev. A</b>  <b>Available on</b> <a href="http://www.ndd.ch">www.ndd.ch</a>	<ul style="list-style-type: none"> <li>• EasyWare V2.25.0.0 (follows V2.24.1.0)</li> <li>• EasyChecker V4.0.2.0 Rev. C with firmware 2.25, 5.10 and 6.7</li> <li>• SmartInstaller V3.1.0.0</li> <li>• Predicted Server V1.17.04</li> <li>• EZ1 Resource Provider V3.0.1 (unchanged)</li> <li>• DeviceCommDll V1.7.5.0</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>Bug fixes and changes:</b></p> <ul style="list-style-type: none"> <li>• Bug #488 Missing parameters in Text Export CSV All Records</li> <li>• Chg #489 Newest DeviceCommDll.dll</li> <li>• Chg #490 New Predicteds V1.17.4 with Zapletal 2003, Quanjer 2012 (GLI) and Falaschetti</li> <li>• Chg #491 Undo Chg #471 (14.10.11 AW Advanced View enabled in online mode 18.10.11 PP 2.23.2.0)</li> <li>• Chg #492 Hidden preference option flag EasyCheckerEna=1 to disable EasyChecker calls for studies</li> </ul> <p><b>EasyChecker V4.0.2.0 Rev. C, unchanged exe, but with modified firmware archive:</b></p> <ul style="list-style-type: none"> <li>• CRA-MRK-02050000.zip</li> <li>• CRA-STD-01040000.zip</li> <li>• CRD-STD-03031400.mot</li> <li>• EZ1-ALD-02150400-M-Res.zip</li> <li>• EZ1-ALD-02150400-P-Res.zip</li> <li>• EZ1-ANT-04020500-P-Res.zip</li> <li>• EZ1-CAA-04050000-P-Res.zip</li> <li>• EZ1-ESS-02220000-P-Res.zip</li> <li>• EZ1-GEC-04030000-P-Res-EU.zip</li> <li>• EZ1-GSK-02220000-P-Res.zip</li> <li>• EZ1-MCL-05050000-P-Res.zip</li> <li>• EZ1-MOT-02140400-M-Res.zip</li> <li>• EZ1-MOT-02140400-P-Res.zip</li> <li>• EZ1-MRK-03070000-P-Res.zip</li> <li>• EZ1-PF2-02170600-P-Res.zip</li> <li>• EZ1-SAV-02220000-P-Res.zip</li> <li>• EZ1-SIM-02220000-P-Res.zip</li> <li>• EZ1-STD-02220000-M-Res.zip</li> <li>• EZ1-STD-02220000-M-Res-Brasil.zip</li> <li>• EZ1-STD-02220000-M-Res-Polish.zip</li> <li>• EZ1-STD-02220000-M-Res-Swedish.zip</li> <li>• EZ1-STD-02220000-P-Res-Polish.zip</li> </ul>

Version	Description
	<ul style="list-style-type: none"> <li>• EZ1-STD-02220000-P-Res-Swedish-Norwegian.zip</li> <li>• <b>EZ1-STD-02250000-M-Res.zip</b></li> <li>• <b>EZ1-STD-02250000-M-Res-Austria.zip</b></li> <li>• <b>EZ1-STD-02250000-M-Res-Polish.zip</b></li> <li>• <b>EZ1-STD-02250000-M-Res-Swedish-Norwegian.zip</b></li> <li>• <b>EZ1-STD-02250000-P-Res.zip</b></li> <li>• <b>EZ1-STD-02250000-P-Res-EE.zip</b></li> <li>• EZ1-STD-05050000-P-Res.zip</li> <li>• EZ1-STD-05050000-P-Res-Polish.zip</li> <li>• <b>EZ1-STD-05100000-P-Res.zip</b></li> <li>• <b>EZ1-STD-05100000-P-Res-EE.zip</b></li> <li>• EZ1-STD-06010000-P-Res.zip</li> <li>• <b>EZ1-STD-06070000-P-Res.zip</b></li> <li>• <b>EZ1-STD-06070000-P-Res-EE.zip</b></li> </ul>
<p><b>V2.24.1.0</b> 02.05.2012</p> <p><b>EasyPrint 2012 Rev. A</b></p> <p><b>Available on <a href="http://www.ndd.ch">www.ndd.ch</a></b></p>	<ul style="list-style-type: none"> <li>• EasyWare V2.24.1.0 (follows V2.24.0.0)</li> <li>• This release was published as first edition of EasyPrint on <a href="http://www.ndd.ch">www.ndd.ch</a></li> <li>• NO EasyChecker</li> <li>• SmartInstaller V3.0.3.0 (follows V3.0.2.0)</li> <li>• Predicted Server V1.13.01 (follows V1.12.04)</li> <li>• EZ1 Resource Provider V3.0.1 (unchanged)</li> <li>• DeviceCommDll V1.7.3.0 (follows V1.7.1.0)</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>Changes:</b></p> <ul style="list-style-type: none"> <li>• Chg #485 The range check for serial numbers (2 * 64k) should be removed</li> <li>• Chg #486 No EasyChecker calls in 'Lite' mode (_DIRECT_PRINTING or _MINUTECLINIC)</li> <li>• Chg #487 Lite Mode with menu 'Switch Language...'</li> </ul>
<p><b>V2.24.0.0</b> 12.03.2012</p> <p><b>EasyWare 2012 Rev.A</b></p> <p><b>Available on <a href="http://www.ndd.ch">www.ndd.ch</a></b></p>	<ul style="list-style-type: none"> <li>• EasyWare V2.24.0.0 (follows V2.23.2.0)</li> <li>• EasyChecker V4.0.2.0</li> <li>• SmartInstaller V3.0.3.0 (follows V3.0.2.0)</li> <li>• Predicted Server V1.13.01 (follows V1.12.04)</li> <li>• EZ1 Resource Provider V3.0.1 (unchanged)</li> <li>• DeviceCommDll V1.7.3.0 (follows V1.7.1.0)</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>Bug fixes and changes:</b></p> <ul style="list-style-type: none"> <li>• Bug 476 Command Line Parameter “/db=” does no more work</li> <li>• Bug 477 EasyWare gets trapped if the user tries to store its database on c:\ -&gt; new command “/r”</li> <li>• Chg 478 The number of ‘scanning ndd devices...’ entries have to be reduced in logfile</li> <li>• Bug 479 EasyWare gets trapped if the user tries to open an EasyWare Pro database</li> <li>• Chg 480 New Predicted Sources V1.13.1 for Firmware V6: <ul style="list-style-type: none"> <li>▪ Pérez Padilla 2001: Spiro, Mexican Workers</li> <li>▪ Pérez Padilla 2006: Spiro, PLATINO study, adults &gt;40</li> <li>▪ Klement, Russia: Spiro</li> <li>▪ Platino (Pérez-Padilla) nun mit NHANES-III auto-kombiniert für Alter &lt; 40 Jahre</li> </ul> </li> <li>• New additional predicteds for Firmware V6: <ul style="list-style-type: none"> <li>▪ Nystad</li> <li>▪ Pérez Padilla 2003: Spiro, Mexican Children</li> <li>▪ Vilozny</li> </ul> </li> <li>• Bug 482 A misformed string in the TechID field of the EasyOne causes a DAO exception during synchronization.</li> <li>• Bug 483 No Langhammer Predicteds for Firmware 5.x</li> <li>• Bug 484 No Roca (Spain) Predicteds for Firmware 2.x</li> </ul> <p><b>EasyChecker V4.0.2.0 unchanged exe, but with modified firmware archive:</b></p> <ul style="list-style-type: none"> <li>• CRA-MRK-02050000.zip</li> </ul>

Version	Description
	<ul style="list-style-type: none"> <li>• CRA-STD-01040000.zip</li> <li>• CRD-STD-03031400.mot</li> <li>• EZ1-ALD-02150400-M-Res.zip</li> <li>• EZ1-ALD-02150400-P-Res.zip</li> <li>• EZ1-ANT-04020500-P-Res.zip</li> <li>• EZ1-CAA-04050000-P-Res.zip</li> <li>• EZ1-ESS-02220000-P-Res.zip</li> <li>• EZ1-GEC-04030000-P-Res-EU.zip</li> <li>• EZ1-GSK-02220000-P-Res.zip</li> <li>• EZ1-MCL-05050000-P-Res.zip</li> <li>• EZ1-MOT-02140400-M-Res.zip</li> <li>• EZ1-MOT-02140400-P-Res.zip</li> <li>• EZ1-MRK-03070000-P-Res.zip</li> <li>• EZ1-PF2-02170600-P-Res.zip</li> <li>• EZ1-SAV-02220000-P-Res.zip</li> <li>• EZ1-SIM-02220000-P-Res.zip</li> <li>• EZ1-STD-02220000-M-Res.zip</li> <li>• EZ1-STD-02220000-M-Res-Brasil.zip</li> <li>• EZ1-STD-02220000-M-Res-Polish.zip</li> <li>• EZ1-STD-02220000-M-Res-Swedish.zip</li> <li>• EZ1-STD-02220000-P-Res-Polish.zip</li> <li>• EZ1-STD-02220000-P-Res-Swedish-Norwegian.zip</li> <li>• EZ1-STD-02240000-M-Res.zip</li> <li>• EZ1-STD-02240000-M-Res-Austria.zip</li> <li>• EZ1-STD-02240000-M-Res-Polish.zip</li> <li>• EZ1-STD-02240000-M-Res-Swedish-Norwegian.zip</li> <li>• EZ1-STD-02240000-P-Res.zip</li> <li>• EZ1-STD-02240000-P-Res-EE.zip</li> <li>• EZ1-STD-05050000-P-Res.zip</li> <li>• EZ1-STD-05050000-P-Res-Polish.zip</li> <li>• <b>EZ1-STD-05090000-P-Res.zip</b></li> <li>• <b>EZ1-STD-05090000-P-Res-EE.zip</b></li> <li>• EZ1-STD-06010000-P-Res.zip</li> <li>• <b>EZ1-STD-06060000-P-Res.zip</b></li> <li>• <b>EZ1-STD-06060000-P-Res-EE.zip</b></li> </ul>
<p><b>V2.23.2.0</b> 21.10.2011</p> <p><b>EasyWare 2011 Rev.C</b></p> <p><b>Available on</b> <b>www.ndd.ch</b></p>	<p>EasyWare V2.23.2.0 follows V2.23.1.0 (2011 Rev. B)</p> <ul style="list-style-type: none"> <li>• EasyChecker V4.0.2.0 (follows V4.0.1.0)</li> <li>• SmartStart V3.0.3 (follows V3.0.2)</li> <li>• Predicted Server V1.12.04 (follows V1.12.2.1)</li> <li>• EZ1 Resource Provider V3.0.1</li> <li>• DeviceCommDll V1.7.1.0 (follows V1.6.8.0)</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>EasyWare V2.23.2.0:</b></p> <ul style="list-style-type: none"> <li>• Bug 467 Changing db dir triggers a db copy action which overwrites an already existing db file'</li> <li>• Chg 471 Advanced View enabled in online mode</li> <li>• Bug 473 Cradle Firmware Conflict Message shall only be shown for Synchronization via Cradle</li> </ul> <p><b>EasyChecker V4.0.2.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 44 Upgrade Problem around Austria 1994' - under certain circumstances Austria 1994 Predicteds are erroneously classified as 'must exist'.</li> </ul> <p><b>EasyChecker V4.0.2.0 with the following firmware files:</b></p> <ul style="list-style-type: none"> <li>• CRA-MRK-02050000.zip</li> <li>• CRA-STD-01040000.zip</li> <li>• CRD-STD-03031400.mot</li> <li>• EZ1-ALD-02150400-M-Res.zip</li> </ul>

Version	Description
	<ul style="list-style-type: none"> <li>• EZ1-ALD-02150400-P-Res.zip</li> <li>• EZ1-ANT-04020500-P-Res.zip</li> <li>• EZ1-CAA-04050000-P-Res.zip</li> <li>• EZ1-ESS-02220000-P-Res.zip</li> <li>• EZ1-GEC-04030000-P-Res-EU.zip</li> <li>• EZ1-GSK-02220000-P-Res.zip</li> <li>• EZ1-MCL-05050000-P-Res.zip</li> <li>• EZ1-MOT-02140400-M-Res.zip</li> <li>• EZ1-MOT-02140400-P-Res.zip</li> <li>• EZ1-MRK-03070000-P-Res.zip</li> <li>• EZ1-PF2-02170600-P-Res.zip</li> <li>• EZ1-SAV-02220000-P-Res.zip</li> <li>• EZ1-SIM-02220000-P-Res.zip</li> <li>• EZ1-STD-02220000-M-Res.zip</li> <li>• EZ1-STD-02220000-M-Res-Brasil.zip</li> <li>• EZ1-STD-02220000-M-Res-Polish.zip</li> <li>• EZ1-STD-02220000-M-Res-Swedish.zip</li> <li>• EZ1-STD-02220000-P-Res-Polish.zip</li> <li>• EZ1-STD-02220000-P-Res-Swedish-Norwegian.zip</li> <li>• EZ1-STD-02240000-M-Res.zip</li> <li>• EZ1-STD-02240000-M-Res-Austria.zip</li> <li>• EZ1-STD-02240000-M-Res-Polish.zip</li> <li>• EZ1-STD-02240000-M-Res-Swedish-Norwegian.zip</li> <li>• EZ1-STD-02240000-P-Res.zip</li> <li>• EZ1-STD-02240000-P-Res-EE.zip</li> <li>• EZ1-STD-05050000-P-Res.zip</li> <li>• EZ1-STD-05050000-P-Res-Polish.zip</li> <li>• EZ1-STD-05080000-P-Res.zip</li> <li>• EZ1-STD-05080000-P-Res-EE.zip</li> <li>• EZ1-STD-06010000-P-Res.zip</li> <li>• EZ1-STD-06050000-P-Res.zip</li> <li>• EZ1-STD-06050000-P-Res-EE.zip</li> </ul>
<p><b>V2.23.1.0</b> 25.05.2011</p> <p>EasyWare 2011</p> <p><b>Available on</b> <a href="http://www.ndd.ch">www.ndd.ch</a></p> <p><b>“Download_01972_xx.exe”</b></p>	<p><b>Follows V2.23.0.0 (2011)</b></p> <ul style="list-style-type: none"> <li>• EasyChecker V4.0.1.0 (follows V3.1.0.0)</li> <li>• SmartStart V3.0.2 (follows V2.7.0.0)</li> <li>• Predicted Server V1.12.2.1 (follows V1.12.2.0)</li> <li>• EZ1 Resource Provider V3.0.1</li> <li>• DeviceCommDll V1.6.8.0 (follows V1.6.5.0)</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>SetupEasyWare:</b></p> <ul style="list-style-type: none"> <li>• Chg 1 Target Dir set back to “EasyWare” as used in earlier versions GDT calls have to be checked for this change.</li> </ul> <p><b>EasyWare V2.23.1.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 448 First Program Start should set English as standard language if no dll’s are available</li> <li>• Chg 449 EasyChecker shall be used via its COM-Interface’ – EasyChecker is installed as independent Setup Project.</li> <li>• Chg 450 New Firmware 2.24 has Roca(Spain) Predicted</li> <li>• Chg 456 Device connects are suppressed while showing About Dialog</li> <li>• Chg 458 Automatic Export of Last Test into XML File – same as ExportGDT into Text Export Dir’ – GDT-Naming.xml – CalChecks are not supported.</li> <li>• Bug 460 TestDialog - CalCheck Limit is still +-3% and must set to +-3.5%</li> <li>• Chg 461 EasyChecker Calls are now logged</li> <li>• Bug 463 Preferences – GC Grading Standard -&gt; QC Grading Standard</li> <li>• Bug 464 Enable EasyChecker for Cradle only connected</li> </ul>

Version	Description
	<p><b>EasyChecker V4.0.1.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 44 EasyChecker is an independent Setup Project' – with HKLM-Path Retrieving or via COM EasyChecker.Application GetTargetDir().</li> <li>• Chg 45 Default firmware archive switched from CSIDL_COMMON_APPDATA to CSIDL_PROGRAM_FILES</li> <li>• Chg 46 EasyChecker is started in Advanced Mode Dialog</li> <li>• Chg 49 New Predicted Austrian 1994 Fingerprint</li> </ul> <p><b>SmartStart V3.0.2.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 12 'requires admin privileges' replaces 'admin rights needed'</li> <li>• Bug 13 Adobe Flash Player isn't detected correctly – there is a difference between Plugin and ActiveX.</li> <li>• Chg 16 Status text with admin part only if needed</li> <li>• Chg 17 New TYPE_TXT for Readme.txt</li> </ul> <p><b>EasyChecker V4.0.1.0 with the following firmware files:</b></p> <ul style="list-style-type: none"> <li>• CRA-MRK-02050000.zip</li> <li>• CRA-STD-01040000.zip</li> <li>• CRD-STD-03031400.mot</li> <li>• EZ1-ALD-02150400-M-Res.zip</li> <li>• EZ1-ALD-02150400-P-Res.zip</li> <li>• EZ1-ANT-04020500-P-Res.zip</li> <li>• EZ1-CAA-04050000-P-Res.zip</li> <li>• EZ1-ESS-02220000-P-Res.zip</li> <li>• EZ1-GEC-04030000-P-Res-EU.zip</li> <li>• EZ1-GSK-02220000-P-Res.zip</li> <li>• EZ1-MCL-05050000-P-Res.zip</li> <li>• EZ1-MOT-02140400-M-Res.zip</li> <li>• EZ1-MOT-02140400-P-Res.zip</li> <li>• EZ1-MRK-03070000-P-Res.zip</li> <li>• EZ1-PF2-02170600-P-Res.zip</li> <li>• EZ1-SAV-02220000-P-Res.zip</li> <li>• EZ1-SIM-02220000-P-Res.zip</li> <li>• EZ1-STD-02220000-M-Res.zip</li> <li>• EZ1-STD-02220000-M-Res-Brasil.zip</li> <li>• EZ1-STD-02220000-M-Res-Polish.zip</li> <li>• EZ1-STD-02220000-M-Res-Swedish.zip</li> <li>• EZ1-STD-02220000-P-Res-Polish.zip</li> <li>• EZ1-STD-02220000-P-Res-Swedish-Norwegian.zip</li> <li>• EZ1-STD-02240000-M-Res.zip (NEW)</li> <li>• EZ1-STD-02240000-M-Res-Austria.zip (NEW)</li> <li>• EZ1-STD-02240000-M-Res-Polish.zip (NEW)</li> <li>• EZ1-STD-02240000-M-Res-Swedish-Norwegian.zip (NEW)</li> <li>• EZ1-STD-02240000-P-Res.zip (NEW)</li> <li>• EZ1-STD-02240000-P-Res-EE.zip (NEW)</li> <li>• EZ1-STD-05050000-P-Res.zip</li> <li>• EZ1-STD-05050000-P-Res-Polish.zip</li> <li>• EZ1-STD-05080000-P-Res.zip (NEW)</li> <li>• EZ1-STD-05080000-P-Res-EE.zip (NEW)</li> <li>• EZ1-STD-06010000-P-Res.zip</li> <li>• EZ1-STD-06050000-P-Res.zip (NEW)</li> <li>• EZ1-STD-06050000-P-Res-EE.zip (NEW)</li> </ul>
<p><b>V2.23.0.0</b> 31.01.2011</p> <p>EasyWare 2011</p> <p><b>Available on</b> <a href="http://www.ndd.ch">www.ndd.ch</a></p>	<p><b>Follows V2.22.0.0 (2010)</b></p> <ul style="list-style-type: none"> <li>• EasyWare V2.23.0.0 (follows V2.22.0.0)</li> <li>• EasyChecker V3.1.0.0 (replaces EasyUpdate V2.2.0.0 and does no more need Admin Rights)</li> <li>• SmartStart V2.7.0 (follows V2.5.0.0)</li> <li>• Predicted Server V1.12.2 (follows V1.10.5)</li> <li>• EZ1 Resource Provider V3.0.1</li> <li>• DeviceCommDll V1.6.5.0 (follows V1.6.4.0)</li> </ul>

Version	Description
<p><b>“Download_01972_02.exe”</b></p>	<ul style="list-style-type: none"> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>EasyWare V2.23.0.0:</b></p> <ul style="list-style-type: none"> <li>• Bug 422 Online-Test FVL, second trial with timeout overwrites first blown test results</li> <li>• Bug 423 DeviceCommDll.dll needs 4s delay for ScreenConnector Plugin Event</li> <li>• Bug 424 ‘NHANES III/Polgar in Frontline and NLHEP Mode’. Additional predicted values will be reset to NO_PAEDIATRIC in case of (Mode = NLHEP    (Mode = FRONTLINE AND Type = DIAGNOSTIC_US)    Type = FRONTLINE_US) AND PedPredRef != NO_PAEDIATRIC</li> <li>• Chg 425 PredAut.dll contains more fixed predicted values and a new concept in filling the Combobox’ is needed</li> <li>• Chg 426 EasyChecker replaces EasyUpdate</li> <li>• Bug 427 EasyWare crashes if About Screen is opened for the first time’ -&gt; DeviceCommDll.dll GetLibraryVersions accesses an unloaded ftd2xx.dll library</li> <li>• Bug 428 Cradle V2.5 and EasyOne V5/V6 is a forbidden combination for synchronisation</li> <li>• Bug 430 DAO Errors are shown behind the Splash Screen</li> <li>• Bug 431 If Create DB fails a lot of error messages are shown</li> <li>• Bug 434 /unregserver is not interpreted correctly</li> <li>• Chg 435 Test Type is shown with number of trials’ – column text “Type (Trials)”</li> <li>• Bug 438 MEF/FEF 25, 50 and 75 are shown twice in Test Dialog</li> <li>• Chg 439 Show Please Wait... and EasyChecker... while Checking Process is running</li> <li>• Bug 440 NLHEP and FEV6 and PEF display is OFF should display %Pred column only</li> <li>• Chg 441 Service Menu Type Change shows message “Please use a ScreenConnector”</li> <li>• Chg 442 Windows XP Style</li> <li>• Chg 443 Auto create pdf after examination</li> </ul> <p><b>EasyChecker V3.1.0.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 35 New Software Name EasyChecker – Project was cloned and cleaned from ‘install’, ‘update’ and ‘setup’ to avoid the need for Admin Rights (Shielded Windows Symbol).</li> <li>• Chg 36 New Parameter /m for Minimal User Interface</li> <li>• Bug 37 Firmware V04.00.04.00 isn’t handled correctly’ In this case a firmware switch should be proposed.</li> <li>• Chg 38 New Firmware 5.8 must be specially handled</li> <li>• Chg 39 Firmware 2.23 must be specially handled</li> <li>• Bug 40 Language is logged in exe path</li> <li>• Bug 41 Firmware 2.22 isn’t updated correctly to 2.23’ - patch for language depending updates</li> <li>• Bug 42 Firmware V04.00.04.00 isn’t updated correctly to 2.23</li> </ul> <p><b>SmartStart V2.7.0.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 6 More reliable method to detect installed software</li> <li>• Chg 7 admin=1 suppresses driver, firmware and flash installation</li> <li>• Bug 8 Restore button stati depending on ini flags</li> <li>• Chg 9 Buttons without living link will be removed from Dialog</li> </ul>
	<p><b>Bundled Firmware (Changes):</b></p> <ul style="list-style-type: none"> <li>• New Firmware EZ1-STD-02230000-M-Res.zip</li> <li>• New Firmware EZ1-STD-02230000-M-Res-Polish.zip</li> <li>• New Firmware EZ1-STD-02230000-M-Res-Swedish-Norwegian.zip</li> <li>• New Firmware EZ1-STD-02230000-P-Res.zip</li> <li>• New Firmware EZ1-STD-02230000-P-Res-EE.zip</li> <li>• New Firmware EZ1-STD-05080000-P-Res.zip</li> <li>• New Firmware EZ1-STD-05080000-P-Res-Po-Ru.zip</li> <li>• New Firmware EZ1-STD-06050000-P-Res.zip</li> <li>• New Firmware EZ1-STD-06050000-P-Res-NE-EU.zip</li> <li>• Old Firmware EZ1-STD-02220000-M-Res.zip</li> <li>• Old Firmware EZ1-STD-02220000-M-Res-Polish.zip</li> <li>• Old Firmware EZ1-STD-02220000-M-Res-Swedish.zip</li> <li>• Old Firmware EZ1-STD-02220000-P-Res-Polish.zip</li> <li>• Old Firmware EZ1-STD-02220000-P-Res-Swedish-Norwegian.zip</li> </ul>

Version	Description
	<ul style="list-style-type: none"> <li>• Old Firmware EZ1-STD-05050000-P-Res.zip</li> <li>• Old Firmware EZ1-STD-05050000-P-Res-Polish.zip</li> <li>• Old Firmware EZ1-STD-06010000-P-Res.zip</li> </ul>
<p><b>V2.22.0.0</b> 06.10.2010</p> <p>Available on <a href="http://www.ndd.ch">www.ndd.ch</a></p> <p>“Download_01972_01.exe”</p>	<p><b>Follows V2.21.0.0 (2010)</b></p> <ul style="list-style-type: none"> <li>• EasyWare V2.22.0.0 (follows 2.21.0.0)</li> <li>• EasyUpdate V2.2.0.0 (no change only new version)</li> <li>• SmartStart V2.5.0.0</li> <li>• Predicted Server V1.10.5 (follows 1.9.3)</li> <li>• EZ1 Resource Provider V3.0.1</li> <li>• DeviceCommDll V1.6.4.0 (follows 1.6.1.0)</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>EasyWare V2.22.0.0:</b></p> <ul style="list-style-type: none"> <li>• Chg #418 Backups of EasyOne’s with new hardware &gt;= V7.5 are supported.</li> <li>• Chg #421 Test Dialog shows detailed information about connected EasyOne and Connection Device.</li> </ul> <p><b>SmartStart V2.5.0.0:</b></p> <ul style="list-style-type: none"> <li>• Chg #9 New Firmware EZ1-STD-05070000-P-Res.zip replaces EZ1-STD-05060000-P-Res.zip</li> </ul>
<p><b>V2.21.0.0</b> 11.06.2010</p> <p>Available on <a href="http://www.ndd.ch">www.ndd.ch</a></p>	<p><b>Follows V2.20.0.0 (2010)</b></p> <ul style="list-style-type: none"> <li>• EasyWare V2.21.0.0 (follows 2.20.0.0 – no changes – new number for CD production only)</li> <li>• EasyUpdate V2.1.0.0</li> <li>• SmartStart V2.5.0.0 (follows 2.3.1.0) – with pre-installing all WHQL approved ndd drivers</li> <li>• Predicted Server V1.9.3</li> <li>• EZ1 Resource Provider V3.0.1</li> <li>• DeviceCommDll V1.6.1.0 (follows 1.6.0.0)</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – WHQL approved (NEW)</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved</li> </ul> <p><b>SmartStart V2.5.0.0:</b></p> <ul style="list-style-type: none"> <li>• Chg #5 ‘Installation of Winqual approved Drivers’</li> <li>• Chg #6 ‘New Firmware EZ1-STD-06020000-P-Res.zip replaces EZ1-STD-06010000-P-Res.zip’</li> </ul>
<p><b>V2.20.0.0</b> 04.05.2010</p> <p>Available on <a href="http://www.ndd.ch">www.ndd.ch</a></p>	<p><b>Follows V2.14.5.0 (2010)</b></p> <ul style="list-style-type: none"> <li>• EasyUpdate V2.1.0.0 – follows V2.0.0.0</li> <li>• Predicted Server V1.9.3 – follows V1.8.3.0</li> <li>• EZ1 Resource Provider V3.0.1</li> <li>• DeviceCommDll V1.6.0.0</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – ndd signed</li> <li>• ndd ScreenConnector for EasyOne (2.6) – WHQL approved (follows 2.4.6) – this driver is pre-installed and also available via Windows Update.</li> </ul> <p><b>EasyWare V2.20.0.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 374 ‘64-Bit Compatibility’ – for EasyWare with ScreenConnector and PictBridge Cradle</li> <li>• Bug 402 ‘(c)ndd 2000-2008’ new range for year 2010</li> <li>• Chg 403 ‘New Predicted Austrian 1994’</li> <li>• Bug 404 ‘Admin cannot change db name’ admins write to HKLM, others write to HKCU. New users read from HKLM if possible else from HKCU.</li> <li>• Chg 405 ‘Detailed Logging Information for Cradles’</li> <li>• Chg 406 ‘nddmed.log and update.log are located in programdata\ndd’ – before it was the same location as the database. DB directory switches no longer force a splitting of these log files.</li> <li>• Chg 407 ‘Dialog Language Selection is shown always on top’</li> <li>• Chg 408 ‘Signed Standard Deviation for ERS/ECC Zapletal’</li> <li>• Chg 409 ‘DB info in main window’s title bar’</li> </ul>

Version	Description
	<ul style="list-style-type: none"> <li>• Bug 410 'Last rec# in EasyOne is not the highest one' leads to wrong corrupt message caused by #392.</li> <li>• Chg 411 'Enhanced logging for GDT Recall'</li> <li>• Bug 412 'Message for corrupt test list removed'</li> </ul> <p><b>EasyUpdate V2.1.0.0:</b></p> <ul style="list-style-type: none"> <li>• Chg 32 'Firmware V3.26 is not detected as up to date' (LuNoKid!)</li> <li>• Chg 33 'Enhanced Upgradeability for Firmware V2.x, V5.x and V6.x'</li> <li>• Bug 34 'EASONE_MAIN_V4 marks always P-Type' – that is a firmware without resources marking PictBridge Devices. Until now EZU detected such hexfiles of type M.</li> </ul> <p><b>Predicted Server V1.9.3.0:</b></p> <ul style="list-style-type: none"> <li>• 'New Predicted Austrian 1994' reloadable</li> </ul>
<p><b>V2.14.5.0</b> 24.03.2010</p> <p><b>Available on</b> <a href="http://www.ndd.ch">www.ndd.ch</a></p>	<p><b>Follows V2.14.4.0 (2010)</b></p> <ul style="list-style-type: none"> <li>• EasyUpdate V2.0.0.0</li> <li>• Predicted Server V1.8.3</li> <li>• EZ1 Resource Provider V3.0.1</li> <li>• DeviceCommDll V1.5.2.0 (follows V1.5.1.0)</li> <li>• ndd USB Cradle for EasyOne (3.4.1.20) – ndd signed</li> <li>• ndd ScreenConnector for EasyOne (2.4.6) – ndd signed</li> </ul> <p><b>EasyWare V2.14.5.0:</b></p> <ul style="list-style-type: none"> <li>• Bug 397 'Progress Dialog – Indexing +1'</li> <li>• Chg 398 'No message about corrupt list if synch is cancelled or an error happened'</li> <li>• Chg 399 'Version info in main window'</li> <li>• Bug 400 'Long curves &gt; 0x4000 cannot be read!' This bug blocks UPG tests for sure. Bugfix in DeviceCommDll V1.5.2</li> <li>• Bug 401 'Corrupt Cradle is wrongly detected!' related to Chg #387 based on 2 criterias: 1) CRC and 2) Version Number. The CRC in different released versions is not testable - like V1.3. So the CRC test is only done for &gt;= 1.4 / &gt;= 2.5 / &gt;= 3.3.</li> </ul>