

Author(s): YBa, AW  
Date: July 2018  
Version: 2

## Printing with EasyOne Air

We are committed to keeping the information presented in this document up-to-date. Please check <https://www.ndd.ch/en/support/download/application-notes.html> for the latest version.

### 1 Scope

This document describes how reports can be printed to a PDF file or to an EasyOne Air compatible printer. Find a list of compatible printers below.


### 2 Printing to an external printer

#### 2.1 Setup

Connect EasyOne Air to the printer. There are two possible setups:

- Direct connection to printer.
- Connection to printer via the cradle.

In the picture below, EasyOne Air is connected to the printer (and battery charger) via the cradle:

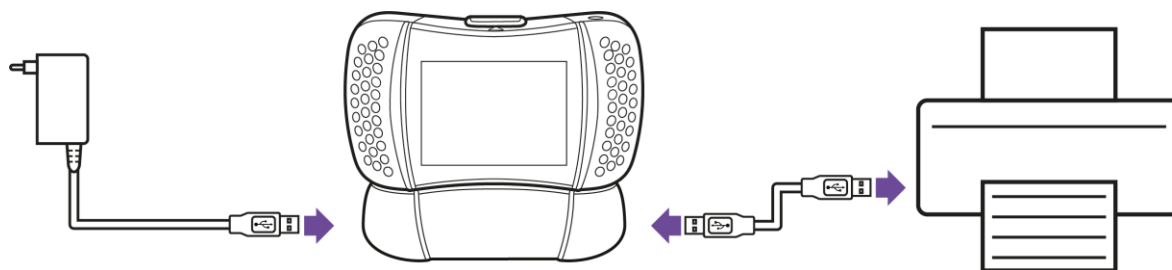

Start the device and select *External printer* under *Tools->Settings->Print*. Configure additional settings such as page size or margins, if desired.

Refer also to the section “setting up the physical components” in the Operator’s Manual [1].

#### 2.2 Procedure

- Select an item under *Reports*. Click *Print*.
- Wait until the report has been printed. Click *OK*.

## 2.3 Printer compatibility

There is a large choice of printers on the market, each supporting different print formats (or languages). In order to print successfully, the printer must support at least one of the following formats:

1. Direct PDF
2. Postscript
3. HP PCL 3 Enhanced protocol
4. HP PCL3GUI (Firmware  $\geq$  V1.10.0)

# Application Note



The table below provides a list of printers<sup>1</sup> that have been tested to date. Note that some printers do not work even though they meet the specifications above.

Manufacturer	Model	Compatible	Comments
Epson	Ink Tank L310, M105, L130, L220.	No	
HP	Color LaserJet CP1515n	Yes	
HP	DeskJet 6940	Yes	Tested black and white only.
HP	LaserJet 500 M551 color	Yes, PDF	Supports Direct PDF printing. Very high printout quality.
HP	HP LaserJet Enterprise M506	Yes, PDF	Black and white printer. Prints PDF via Postscript in grayscale.
HP	OfficeJet 150 Mobile All-in-One	Yes, PCL	
HP	OfficeJet 200 Mobile	Yes	PCL3 capable. Very fast. Color printer.
HP	OfficeJet 8100	Yes, PCL3GUI	
HP	OfficeJet 8210	Yes, PDF	
HP	OfficeJet H470	Yes, PCL	High quality, compact printer.
HP	OfficeJet Pro 251dw	Yes, PDF	Supports Direct PDF printing. Very high printout quality.
HP	OfficeJet Pro 6230	Yes, PCL3GUI	PCL color print.
HP	PageWide Pro MFP 477dw	Yes, PDF	Supports Direct PDF printing. Very high printout quality. Very fast.
HP	DeskJet 2130 All-in-One series, Deskjet Ink Advantage 2020, 2130, 2700 series	Yes <sup>2</sup> , PCL3GUI	

---

<sup>1</sup>This list is purely informative, n d d is not affiliated with any of the printer manufacturers mentioned in this list.

<sup>2</sup> Requires firmware version V1.10.0 or later

HP	DeskJet 1110	Yes <sup>3</sup> , PCL3GUI	HP PCL GUI format I supported for firmware $\geq$ V1.10.0
HP	DeskJet 3639	Yes <sup>3</sup> , PCL3GUI	HP PCL GUI format I supported for firmware $\geq$ V1.10.0
HP	Laser Jet 4250n	Yes, PCL	
OKI	C531dn	Yes, PCL	Supports Direct PDF printing. Very high printouts quality. Output scaled 95% of original size.
TSC	ALPHA-4L <sup>4</sup>	Yes	Thermal printer, 112mm (4.4") wide paper roll, 203 DPI, optionally battery operated
TSC	DA200 <sup>4</sup>	Yes	Thermal printer, 112mm (4.4") wide paper roll, 203 DPI

---

<sup>3</sup> Requires firmware version V1.10.0 or later

<sup>4</sup> Requires firmware version V1.7.0 or later

## 3 Printing to a PDF file

EasyOne Air can print reports to a PDF file on the *exchange* drive on the device. The printouts can then be accessed via a PC when the spirometer is connected via USB. Note that no special software is needed for this function.

### 3.1 Setup

Start the device and select *PDF printer* under *Tools->Settings->Print*.

### 3.2 Procedure

- Select an item under *Reports*. Click *Print*.
- Wait until the PDF file has been created. Click *OK*.
- Connect the device to a PC (via cradle or directly). Usually, a pop-up window opens displaying the *exchange* drive.
- If no pop up window appears, try opening the *exchange* drive manually. E.g. on Windows 8, select *This PC->EasyOne Air->exchange*.
- Select the desired PDF printout. Hint: the file name contains the date and time of the report.

## 4 Printing to a thermal printer

EasyOne Air can print reports on thermal printers.

### 4.1 Setup

Start the device and select *Thermal printer* under *Tools->Settings->Print*.

Connect the printer to EasyOne Air via USB. The printer uses a mini USB connector, so one of the following solutions is required:

Solution 1: use a USB 2.0 **OTG** cable, type Micro A to Mini B:



Solution 2: use the USB A cable delivered with EasyOne Air together with the following adapter (purchased separately):

Micro USB 2.0 OTG adapter cable – USB A female to Micro USB B male.



## 4.2 Procedure

- Select an item under *Reports*. Click *Print*.
- Wait until the report has been printed. Click *OK*.

## 5 References

[1] ndd. 2017. *EasyOne Air Operator's Manual*. Zürich.